

ALCOCHECK® BREATH ALCOHOL SCREEN TRAINING



ALCOCHECK®
✓

AlcoCheck® Breath Alcohol Screen disposable device



FEATURES

- Accurate and reliable results in 2 minutes
- FDA 510(k) cleared for over-the-counter (OTC) use
- Compact, plastic tube construction allows test to be carried anywhere
- Individually packaged
- Pass/fail design detects alcohol at different threshold levels
- Available in three threshold levels – .02, .05, .08

ALCOCHECK®
✓

IMPORTANT: See product insert for complete instructions and limitations of use.

PROCEDURE: Step 1


- 1** Wait 15 minutes after last alcoholic intake or drink a glass of water before testing.



Bring the pouch to room temperature (15-30°C or 59-86°F) before opening. Remove the screen from the sealed pouch. To avoid contamination, do not touch the mouthpiece.

PROCEDURE: Step 2

- 2** Hold the middle of the Breath Alcohol Screen. Firmly squeeze the screen to break the inner glass tube. Perform the screen as soon as possible to avoid the affects of humidity. Do not crush or bend the screen. Refer to the picture at right.

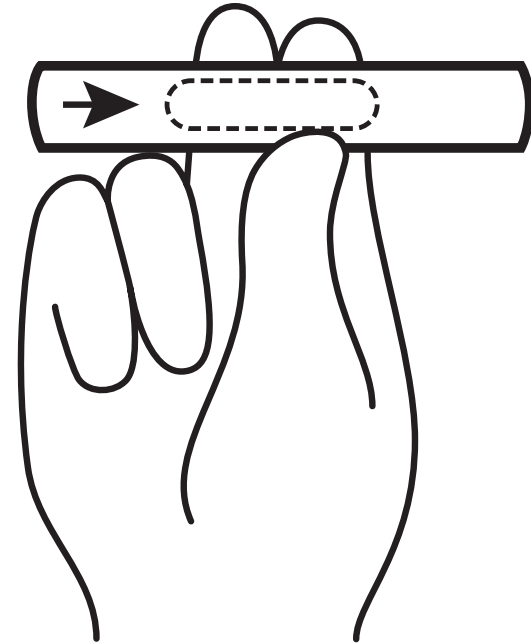


PROCEDURE: Step 3

3

Breathe into the Breath Alcohol Screen.

- Hold the screen vertically with the mouthpiece pointing up so that the crystals fall to the end of the device. Take a deep breath. Blow hard into the mouthpiece, in the same direction as the arrows on the device. Blow hard into the screen for 12 seconds without taking a breath.

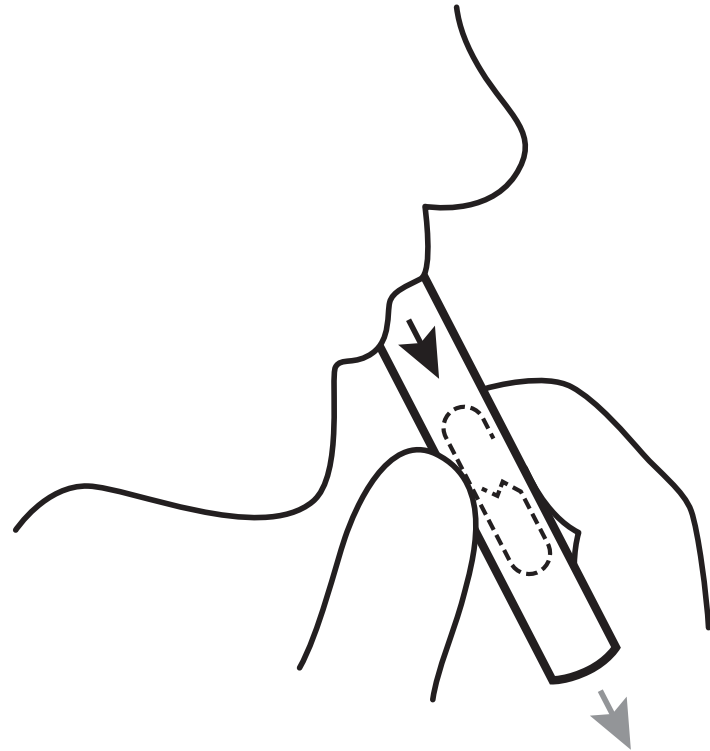


Note: Failure to blow hard, or for 12 seconds, may cause false results. Do not inhale while blowing into the screen. Refer to the picture above.

- Shake the screen slightly to distribute the crystals evenly in the screen window.


PROCEDURE: Step 4 – Interpret Results

4 Read results after 2 minutes by comparing the color of the majority of crystals to the color chart on the screen label. Do not read results after 15 minutes.

POSITIVE: The majority of the crystals are the same color as either color block, or more green-blue than the two color blocks on the screen label. The relative BAC is at or above the cut-off level of the screen.

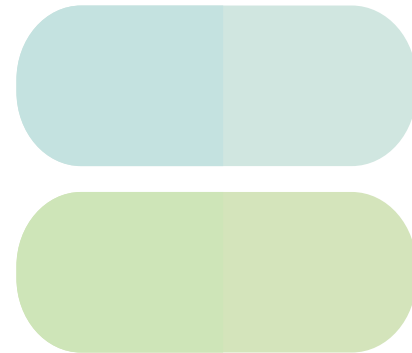

NEGATIVE: The majority of the crystals are more yellow than the cut-off color block on the screen label. The relative blood alcohol content (BAC) is below the cut-off level of the screen. The crystals may be lighter than the crystals from an unused screen.

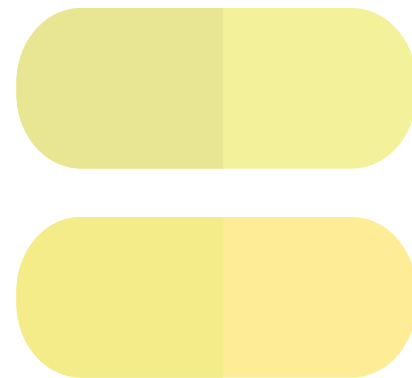

ALCOCHECK[®]

The logo for ALCOCHECK features the brand name in a bold, serif font. A horizontal line underlines the text, which ends in a checkmark symbol on the right side.